



Fitting Instructions

YewdaleKestrel®


Intravenous Cubicle Track (IV200, IV400)

Part no: 1004


What you'll need



Screwdriver

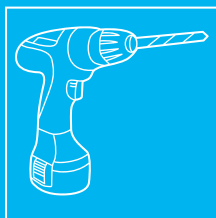


Drill

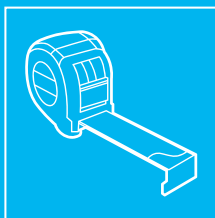


Measuring Tape



Pencil

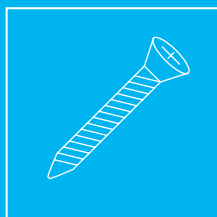


Screws

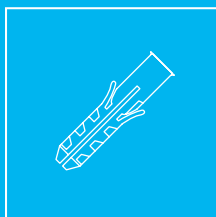


Wall Plugs


What's in the box (IV200)

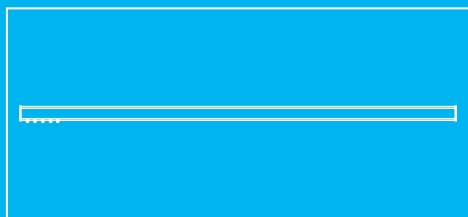


Pre-assembled Track

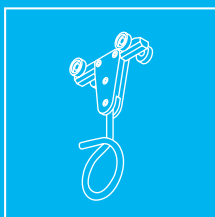


Locking Trolley

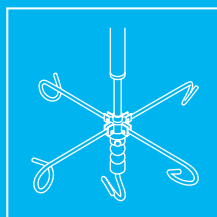


Telescopic Carrier Tree

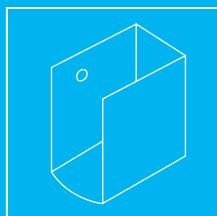


Wall Shoe

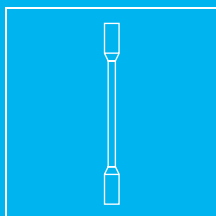


Ceiling Hanger
(optional if L shape or
over 3000mm wide)

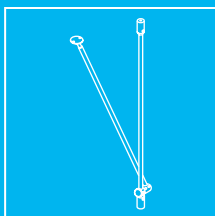


V Hanger
(optional with no lateral
support and is over
3000mm wide)

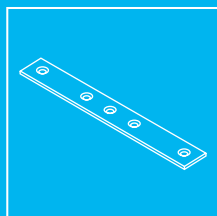


Crossover/T Joints

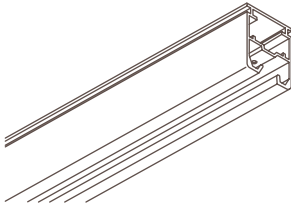

210W
Heavy Duty IV Track

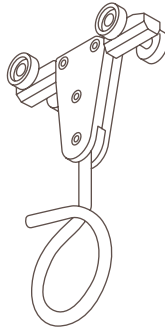

233
Locking Trolley

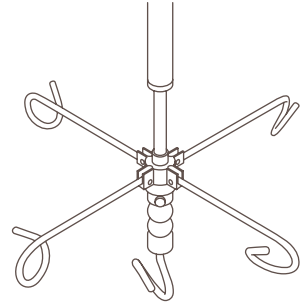

234W
Telescopic Carrier Tree


Loading Figures for IV200

Part No	Description	Design Load
210W	Heavy duty IV track	80kg at each fixing point, therefore the capacity of the track is 80kg x the number of fixings, with load evenly distributed. Fixings should be at 1200mm maximum. If point loads exceeding 25kg (eg. heavy equipment) are anticipated, fixing is recommended at 800mm centres.
233	Locking Trolley	40kg
234W	Telescopic Carrier Tree	Central Hook: 8kg, Folding Hooks: 3kg each. Total loading: 20kg


What you'll need



Screwdriver

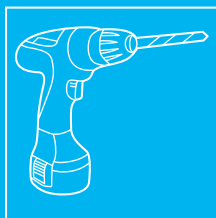


Drill

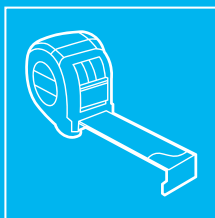


Measuring Tape



Pencil

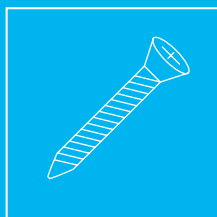

Screws

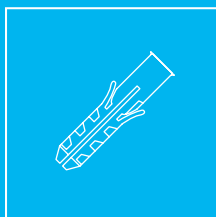


Wall Plugs


What's in the box (IV400)




Pre-assembled Track

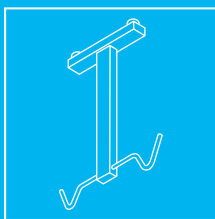


2-Point IV Carrier

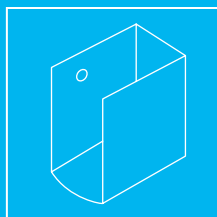


Wall Shoe

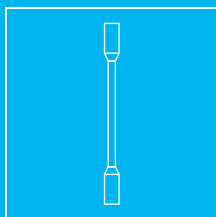


Ceiling Hanger
(optional if L shape or
over 3000mm wide)

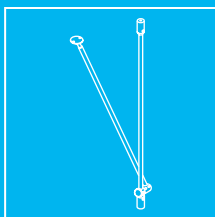


V Hanger
(optional with no lateral
support and is over
3000mm wide)

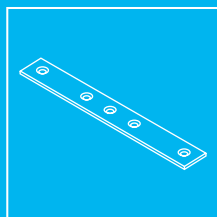

Crossover/T Joints

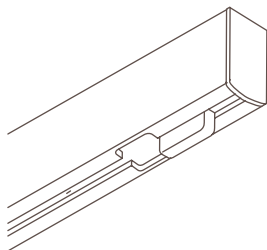

400W
Medium Duty IV Track

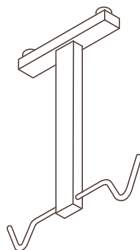

430W
2-Point IV Carrier

Loading Figures for IV400

Part No	Description	Design Load
400W	Medium Duty IV Track	10kg at maximum span 2500mm
430W	2-Point IV Carrier	6kg (3kg per hook)



Warning

All instructions must be read before installing.



Fitting Time

Approx. 20 minutes


Yewdale Movatrack® Technical Book
Further information can be found in the full Yewdale Movatrack® Technical Book.
Please request a copy if required.

Intravenous Curtain Track

Instructions

Before Installation

Unpack all of the track and components and ensure that all of the items are included as per the delivery note. Using the dimensions on the drawing or the agreed track position, mark the position of the track on the ceiling. Ensure adequate support. If fitting to a false ceiling it will be necessary to add timber supports above the ceiling securely supported to the slab above.

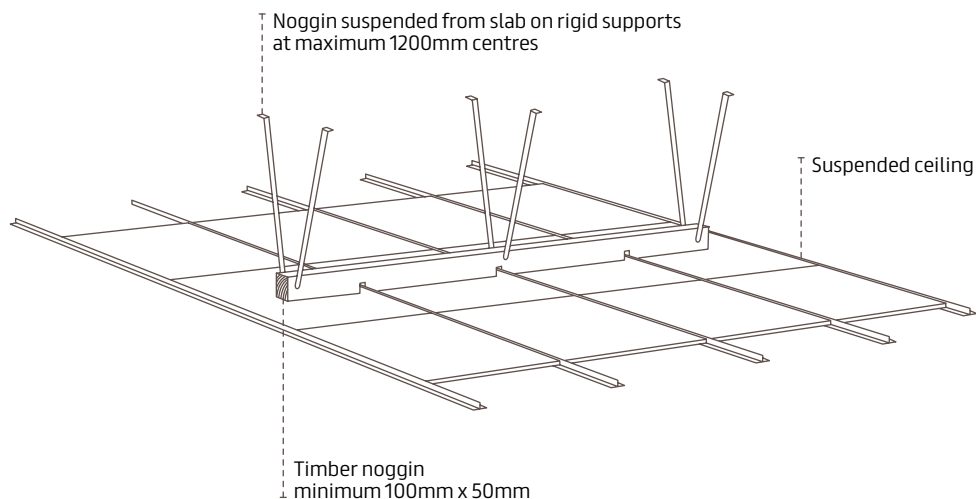


Noggin should be notched to ensure there is no gap between the underside of boggin and the back of the ceiling tile.


Intravenous Curtain Track - IV200

Instructions


- 01** Mark the position of the wall shoes and fix them using suitable screws.

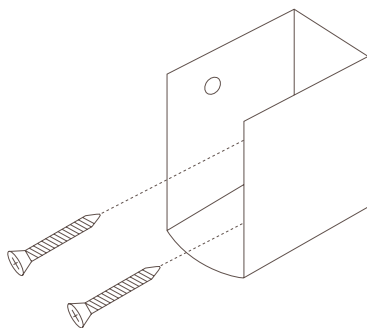


- 02** Assemble each section of the track on a clean floor using track inner joiners where applicable.

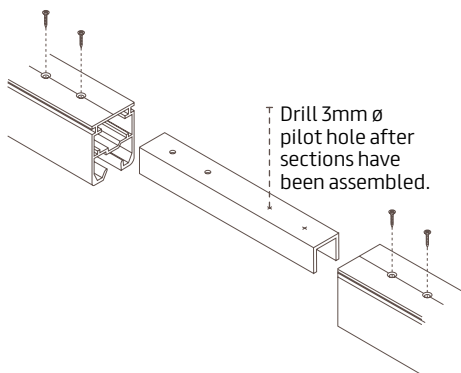

- 03** Lift the first section of the track up into the wall shoes and secure with the screw provided. Make sure to support the track at the bend with a suitable temporary support ensuring that the track is level.

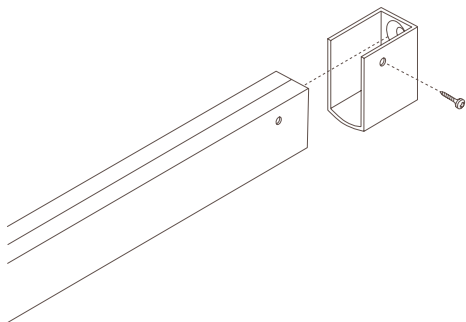


- 04** Remove the end caps and load the locking trolleys onto the track. Replace end caps with the screw provided.

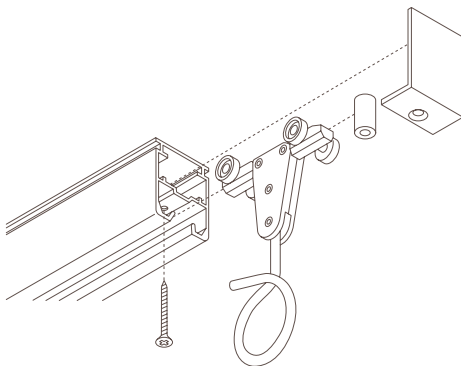


Intravenous Curtain Track - IV200

Instructions


- 05** Attach the telescopic carrier trees and pump carriers on the locking trolleys, ensuring they run freely. Press the button on the carrier to extend up and down.

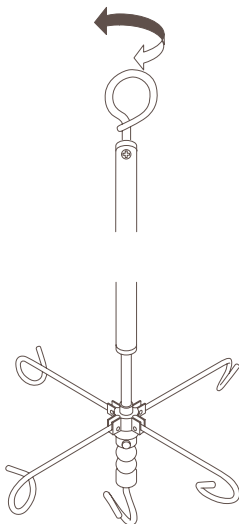

- 06** For electric pump carriers, ensure the locking trolleys supports the cable from the wall connection to where the IV pump carrier hangs. The cable should be supported at every metre.

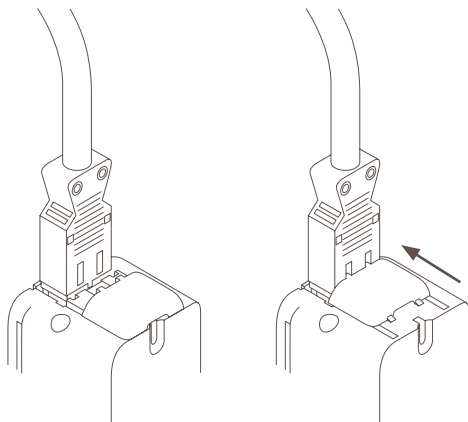


- 06** The cable runs through the loop in the locking trolleys and is secured in place using a cable tie. Any loose ends of the cable tie should be removed. Finally check that the cable moves freely back and forth with the carrier tree and plug in.

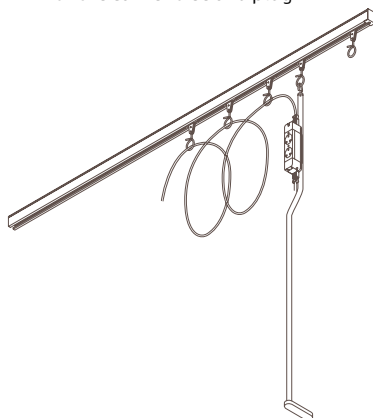


Intravenous Curtain Track - IV400

Instructions

- 01** The IV400 is normally supported between existing YewdaleMovatrack® cubicle track installations running parallel with the bed head wall around 900mm or 1000mm from the wall.

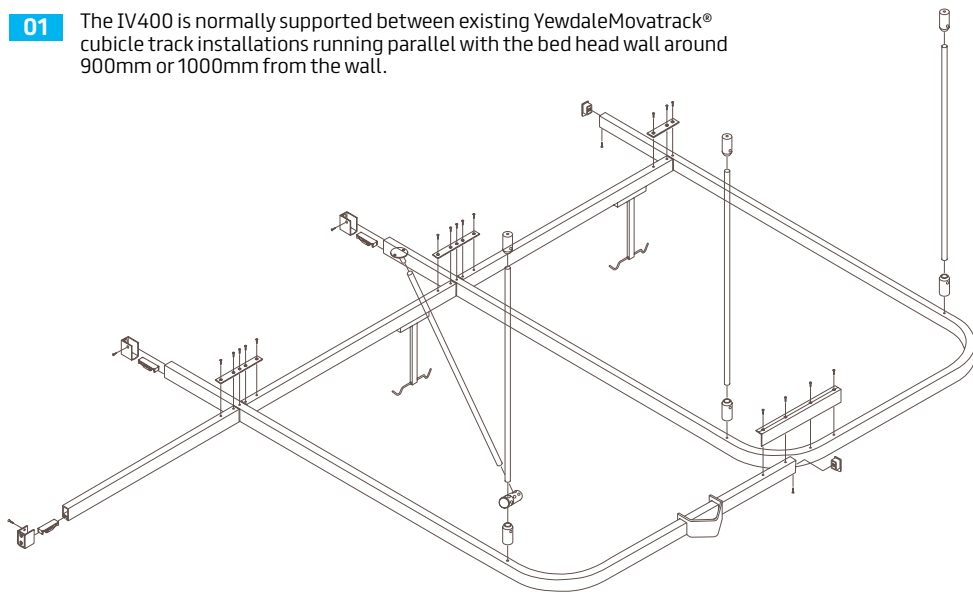


- 02** Support the track with T-joints or crossover (153W and 154W) to the cubicle track and secure to the wall using 123W.

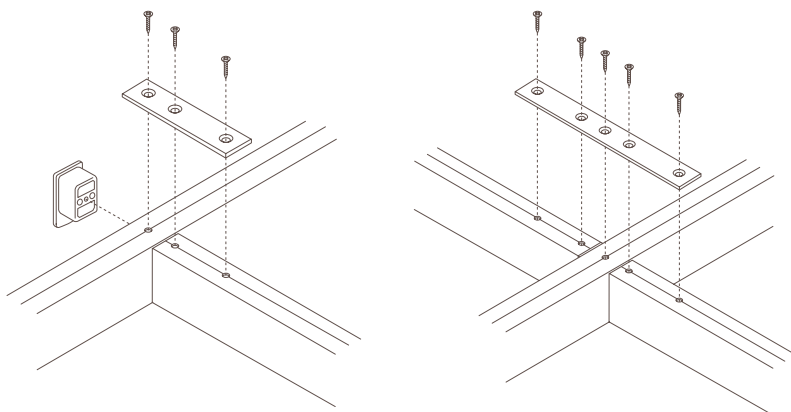


Intravenous Curtain Track - IV400

Instructions

- 03** Once the track is secure, slide the curtain removal point to one side and insert the IV 2 Point Carrier (430W).

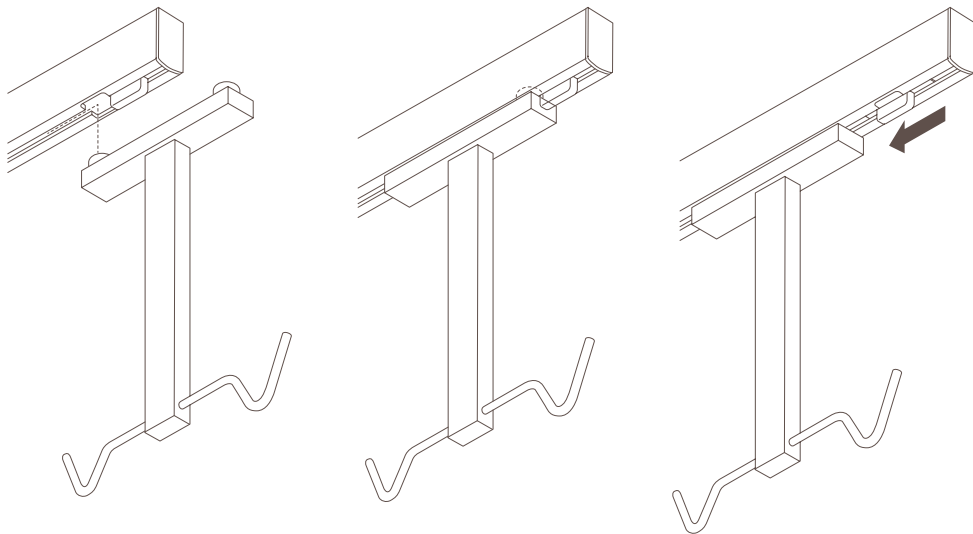

- 04** Ensure this runs freely in the track before use.

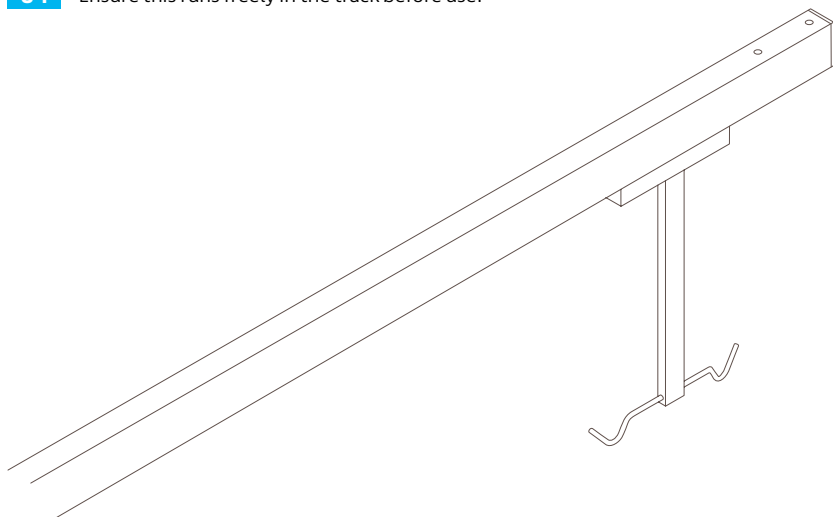



Head Office:

Enterprise Way,
Wickford, Essex, SS11 8DH

Call: +44 (0)1268 570900

Email: sales@yewdale.co.uk

For more information on all
Yewdale products, please call
the sales department.

