




Fitting Instructions

YewdaleMovatrack[®]
Movagate


What you'll need



Screwdriver

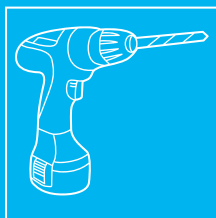


Drill

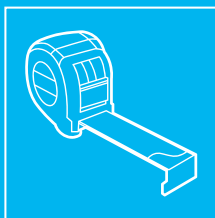


Measuring Tape




Pencil

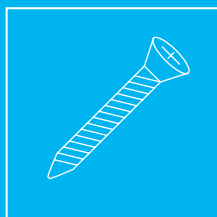

Screws

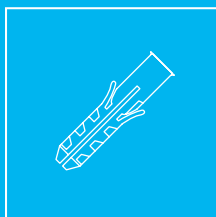


Wall Plugs

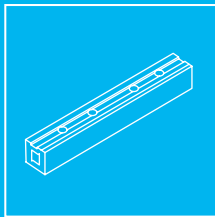

Inner Track Joiners

What's in the box

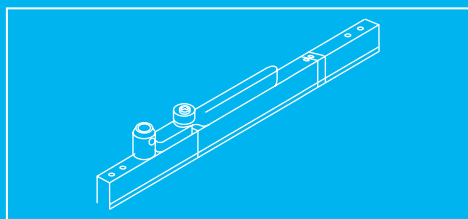

Movagate



Warning

All instructions must be read before installing.




Fitting Time

Approx. 2 - 2.5 hours

Yewdale Movatrack® Technical Book
Further information can be found in the full Yewdale Movatrack® Technical Book.
Please request a copy if required.

Movagate Instructions

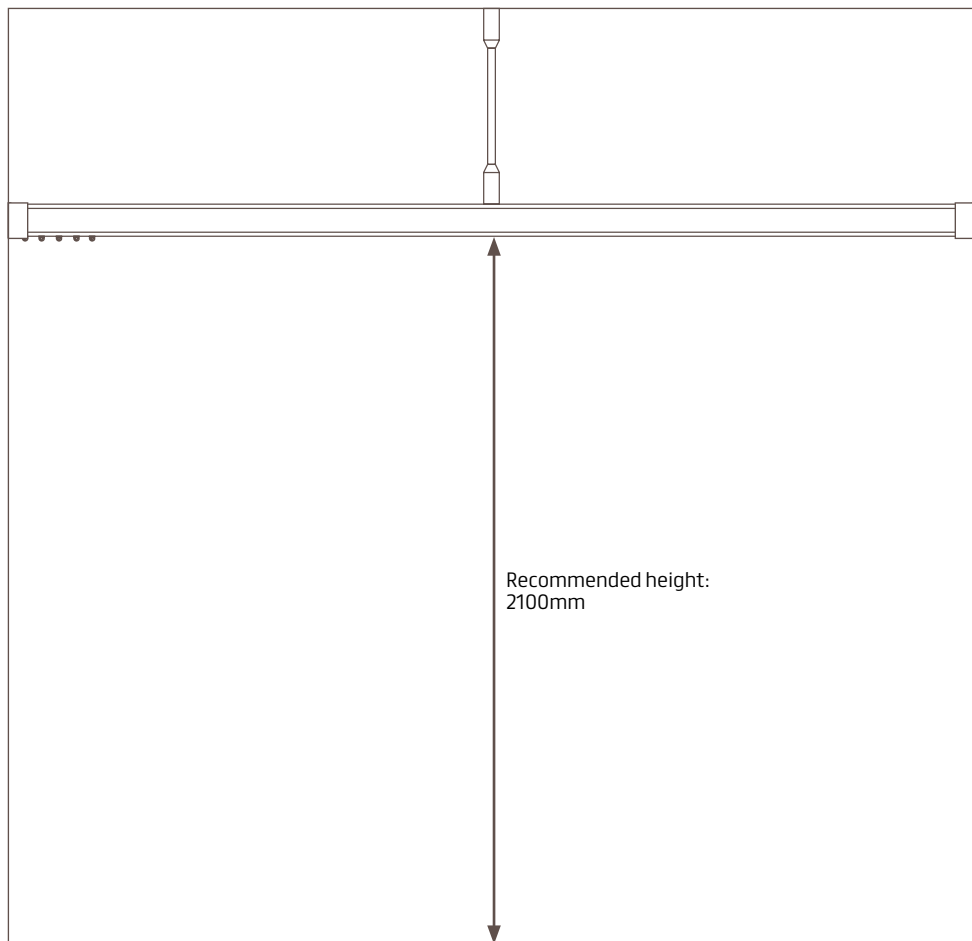


Before Installation

Unpack all of the track and components and ensure that all of the items are included as per the delivery note. Using the dimension on the drawing or the agreed track position, mark the position of the wall shoes on the wall. All dimensions are to the centre of the track.

The HTM66 recommended fitting height is 2100mm from the floor to the underside of the track.

Movagate

Instructions

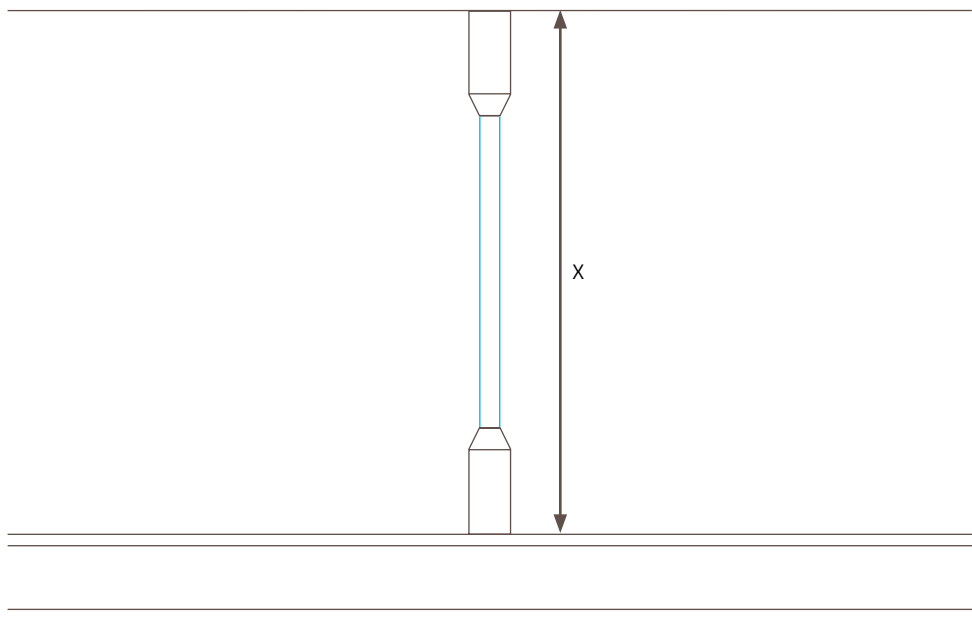


Before Installation

For the correct tube length, you will need to minus (-) 30mm from the overall length of the track to the ceiling (x).


Each bend must be supported by a 313 hanger assembly no further than 500mm from either leg of the bend.

At no point should the track span more than 3000mm without a support. If the track has a join, ensure the track is suspended within 150mm from the join. If the track has no lateral support and spans more than 3000mm, please ensure a V-hanger is used.

Movagate Instructions

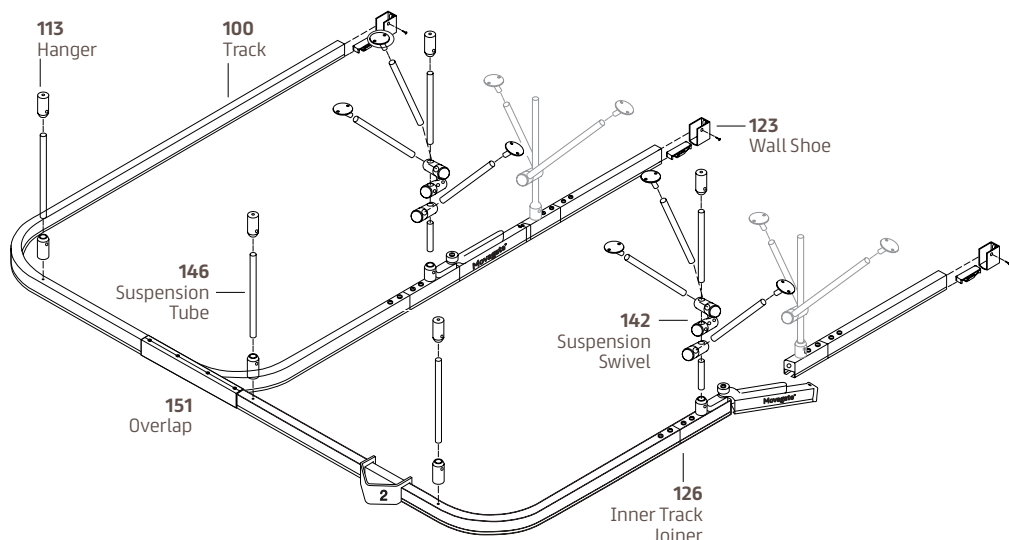


- 01** The Movagate system is designed to be retro-fitted once the hoist tracks are all in place and the standard cubicle track has all been installed to the desired layout. Position gate with strap in position of hinge arm. Once the hook is in position hold gate against track and mark.

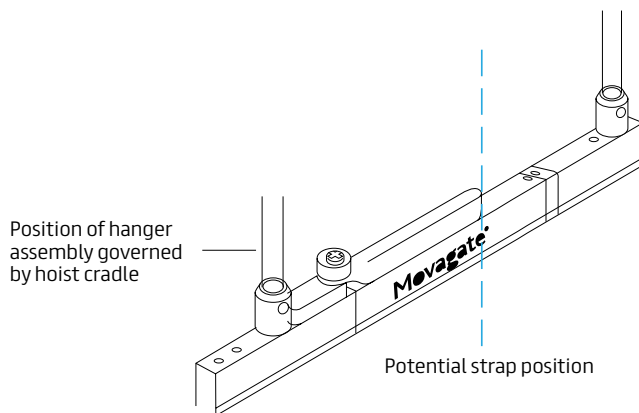


Movagate Instructions

- 02** Attach the Movagate to the movatrack using inner track inner joiners.

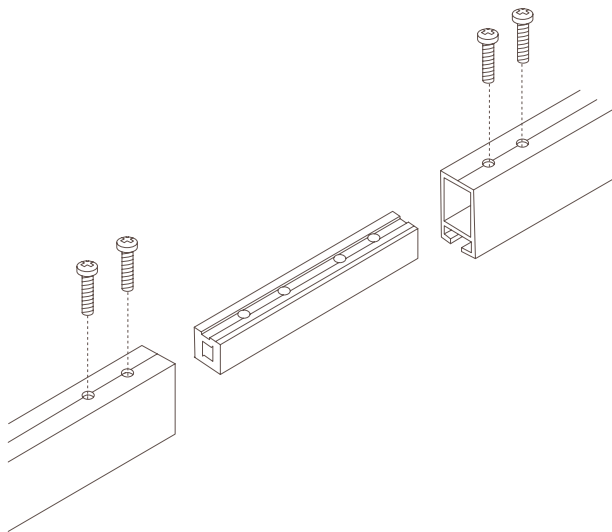


- 03** Each bend must be supported by a 313 hanger, with assembly no further than 500mm from either leg of the bend.

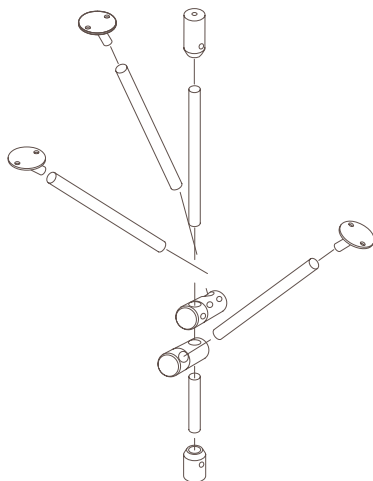


Movagate

Instructions

- 04** Using a plumb line, mark where hanger assembly will fix to ceiling and track.

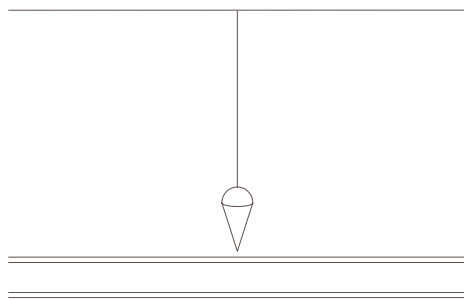


- 05** For V-suspension. When assembling the 313 hanger, ensure that the K113 V-hanger is located on the suspension tube and is rest on the lower 113 hanger which is fixed to the track.

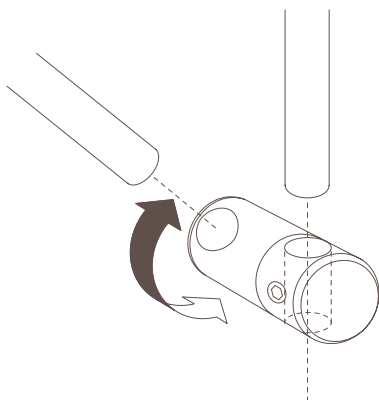


Movagate Instructions

- 06** Once installation is complete, ensure that the gliders run smoothly across any joins and adjust any misalignments accordingly.

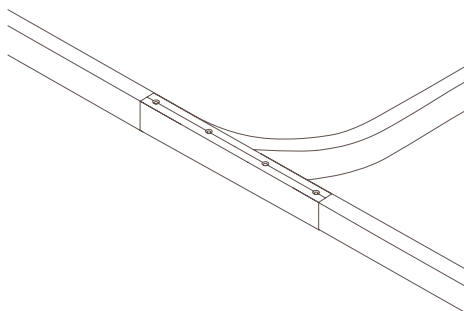

-
- 07** Ensure the Movagate opens and closes smoothly before use. To adjust and retighten V hangers to ensure operation works well with the roller runners in track.

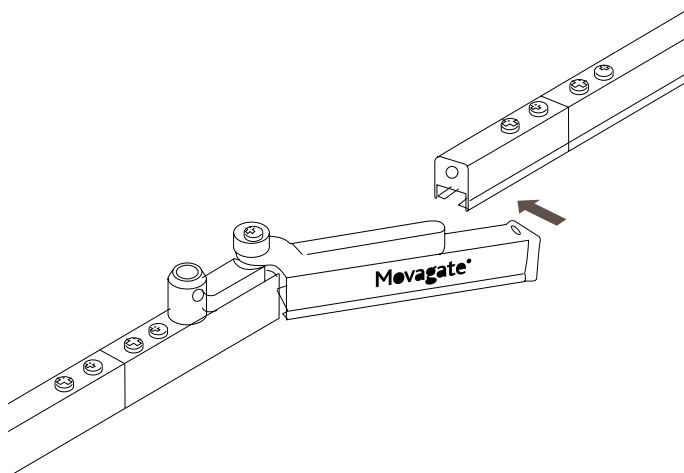



Head Office:

Enterprise Way,
Wickford, Essex, SS11 8DH

Call: +44 (0)1268 570900

Email: sales@yewdale.co.uk

For more information on all
Yewdale products, please call
the sales department.



yewdale **Defiant**[®]
FABRIC AND BLIND SYSTEMS

yewdale **Kestrel**[®]
MAGNETIC ANTI-LEGATURE SYSTEM

yewdale **Movatrack**[®]
CURTICLE AND TRACK SYSTEMS

yewdale **Harrier**[®]
CURTICLE AND SHOWER CURTAINS

yewdale **Vitesse**[®]
CURTAIN TRACK SYSTEMS